




**1** Pour the liquid into the container (max 4ml).

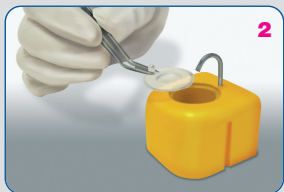

**2** Position the membrane.

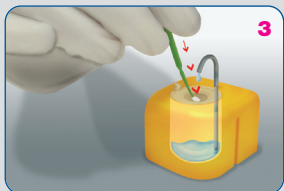

**3** Press gently on the membrane. Hy-Drop releases the liquid drop by drop.

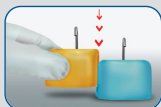

Can be interconnected.

US. Pat. 6267270  
DE Pat. 19737749  
EP Pat. 1006852

**Sterilize in autoclave at 134 °C / 273 °F**



English

### Attention

To avoid cross contamination:  
change membrane after every patient.

Español

### Atención

Para evitar cualquier contaminación cruzada:  
sustituir la membrana después de cada paciente

Italiano

### Attenzione

Per evitare una contaminazione crociata:  
cambiare la membrana dopo ogni paziente.

Deutsch

### Achtung

Um eine Kreuz-Kontamination zu verhindern:  
nach jedem Patient Membrane wechseln.

Français

### Attention

Pour éviter une contamination croisée:  
changer la membrane après chaque patient.

Nederlands

### Waarschuwing

Om crossinfectie te vermijden:  
na elke patient membraam verwisselen.

Português

### Atenção

Para evitar qualquer contaminação cruzada:  
substituir a membrana depois de cada paciente.

Norsk

### Viktig

For å unngå Krysskontaminasjon:  
byt membran etter hver pasient.

Dansk

### Bemærk

For at undgå smitteoverførsel:  
skift membran efter hver patient

Suomi

### Varning

För att undvika korskontaminering:  
membranbyte skall ske efter varje avslutad behandling.

Svenska

### Huom

Kontaminaatio estämiseksi:  
vaihdä kalvo joka potilaan välillä.

## INDICATION OF CHEMICAL RESISTANCE

|   |   |
|---|---|
| Alcohol 70%                                   | ☺ |
| Aliphatic hydrocarbons (n-Butane, Iso-Octane) | ☺ |
| Carbon Tetrachloride                          | ☺ |
| Chlorinated hydrocarbons                      | ☺ |
| Ethanol                                       | ☺ |
| Ethoxyethanol                                 | ☺ |
| Hydrochloric Acid (20%)                       | ☺ |
| Hydrogen peroxide                             | ☺ |
| Hydrolytic stability                          | ☺ |
| Mineral lubricants                            | ☺ |
| Phosphoric acid 85%                           | ☺ |
| Sodium hypochlorite 14%                       | ☺ |
| Strong alkalis (sodium hydroxide 10%)         | ☺ |
| Trichloroethane                               | ☺ |
| Weak alkalis                                  | ☺ |
| Weak mineral acids                            | ☺ |
| Aromatic hydrocarbons                         | ☹ |
| Aromatic solvents (Benzene, Toluene)          | ☹ |
| Perchloroethylene                             | ☹ |
| Strong mineral acids (sulphuric acid 20%)     | ☹ |
| Strong organic acids                          | ☹ |
| Tincture of Iodine                            | ☹ |
| Trichloroethylene                             | ☹ |
| Weak organic acids (acetic acid 20%)          | ☹ |
| Acetone                                       | ☹ |
| Bonding agent                                 | ☹ |
| Chloroform                                    | ☹ |
| Dichloromethane 100%                          | ☹ |
| Methyl Ethyl Ketone                           | ☹ |
| Oxidising acids                               | ☹ |




Suitable



Suitable only  
with daily cleaning




Not suitable

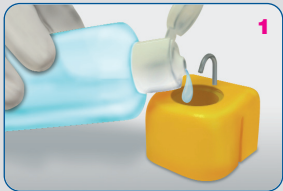


**1** Pour the liquid into the container (max 4ml).



**2** Position the membrane.

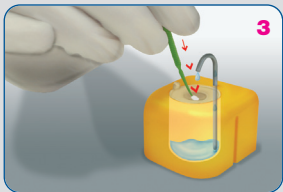

**3** Press gently on the membrane. Hy-Drop releases the liquid drop by drop.

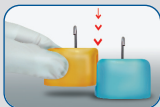

Can be interconnected.

US. Pat. 6267270  
DE Pat. 19737749  
EP Pat. 1006852

**Sterilize in autoclave at 134 °C / 273 °F**



Ελληνικά

**Προσοχή**

Για να αποφύγετε διασταυρούμενη μόλυνση: Αλλάξτε μεμβράνη μετά από κάθε ασθενή.

Polski

**Uwaga**

W celu uniknięcia zakażenia krzyżowego: zmieniać błonę po każdym pacjencie.

Русский

**Внимание**

Во избежание заражения: заменяйте мембрану после каждого пациента.

Český

**Pozor**

Aby nedošlo ke kontaminaci, po každém pacientovi membránu vyměňte.

Magyar

**Figyelem**

A keresztszennyeződés elkerülése végett: cserélje ki a membránt minden beteg után.

Türkçe

**Dikkat**

Çapraz kontaminasyonu önlemek için: her hastadan sonra membranı değiştirin.

한국어

**ご注意!**

相互汚染を防ぐために、各患者の処理後にはダイアフラムを交換してください。

中文

**注意**

为了避免交叉污染: 请在每位患者用过后再更换膜。

日本語

**주의**

교차오염을 피하기 위해: 각 환자를 치료한 후 막을 교체하십시오.

عربي

**مہینت**  
لعباشتملا شولتلا بنجتل :  
ضن یرم لك دعب ءاش غلا لادبتمسا بجي

**INSTRUCTIONS FOR USE**



See instructions

**Please read carefully and keep for future reference.**

**Hygienic liquid dispenser**

**POLYDENTIA** <sup>SA</sup>   
Swiss Products for Fine Dentistry



Only for professional use

**Made in Switzerland by Polydentia SA** • CH - 6805 Mezzovico  
Tel. +41 (0)91 946 29 48 • Fax +41 (0)91 946 32 03  
www.polydentia.com • info@polydentia.com

**Order Form:**

| Description                            | Quantity |
|--|----------|
| REF 5411/Y Hy-Drop yellow, 5 membranes |          |
| REF 5411/O Hy-Drop orange, 5 membranes |          |
| REF 5411/G Hy-Drop green, 5 membranes  |          |
| REF 5411/V Hy-Drop violet, 5 membranes |          |
| REF 5411/B Hy-Drop blue, 5 membranes   |          |
| REF 5415 5 membranes, refill pack      |          |

Address Dental Dealer

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

Stamp

\_\_\_\_\_

\_\_\_\_\_

Signature, Date

\_\_\_\_\_

\_\_\_\_\_

**POLYDENTIA** <sup>SA</sup>   
Swiss Products for Fine Dentistry

www.polydentia.com • info@polydentia.com