



**Prepare teeth for bonding with Fiber-Bond**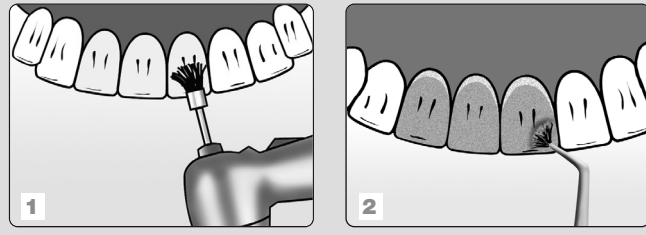


**Coat teeth with Fiber-Bond**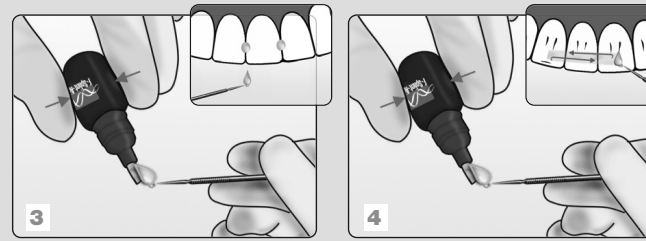

**Prepare the fiber-glass strip**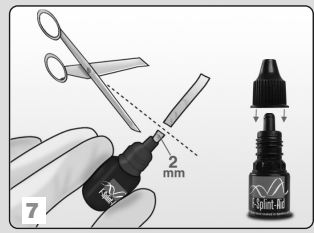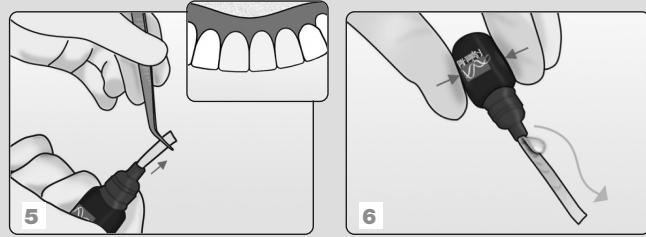

**Place the fiber-glass strip on the teeth, light-cure and finish**

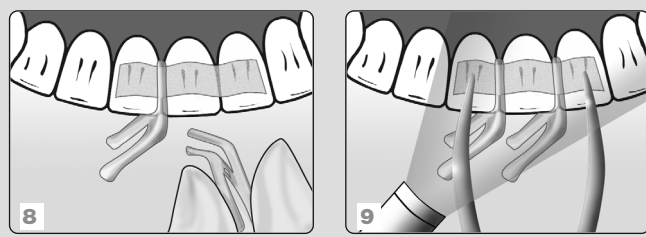**English**

**F-Splint-Aid & Slim**

Fiberglass strip soaked in Fiber-Bond agent. Suitable for 4-7 applications. Indicated for splinting of traumatically or periodontally loose anterior teeth, re-implantation of totally luxated anterior teeth (original extracted tooth or built up with composite). F-Splint-Aid & Slim is also suitable to reinforce temporary bridges or for post-orthodontic retainers.

**INSTRUCTIONS FOR USE**

- Steps 1 to 2: Prepare teeth for bonding with Fiber-Bond**
1. Clean all involved teeth with a non-fluoride paste and place a rubber dam.
  2. Etch the lingual/palatal and interdental surfaces with Fiber-Etch (REF 5991).

**Steps 3 to 4: Coat teeth with Fiber-Bond**

3. Extremely loose teeth can be fixed temporarily, by applying a small amount of Fiber-Bond or Fiber-Flow (REF 5991) interdentally, from the buccal side.
4. Squeeze the F-Splint-Aid & Slim bottle to saturate an applicator with Fiber-Bond. Coat the lingual/palatal and interdental surfaces with Fiber-Bond.

- Steps 5 to 7: Prepare the glass fiber strip**
5. Pull the appropriate strip length from the F-Splint-Aid & Slim bottle with tweezers.
  6. Squeeze the bottle to well saturate the glass fiber strip with Fiber-Bond.
  7. Cut the soaked strip. Leave at least a 2 mm long strip for next use. Close F-Splint-Aid & Slim bottle tightly after use, in order to preserve the product.

**Steps 8 to 10: Place the glass fiber strip on the teeth, light-cure and finish**

8. Place strip onto the lingual/palatal surface and hold it in place with the application clips (Clip&Splint).
9. Keep in place the ends of the strip with tweezers while polymerizing each interproximal space (30-40 sec.).
10. Any rough areas can be smoothed with a layer of Fiber-Flow (REF 5991). The splint is then trimmed and polished. Remove the rubber dam and check the occlusion.

**Important:** Instruct your patient about proper cleaning technique and prophylaxis.

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
**Quick Guide**

*A world first!*

**READY TO USE SOLUTION FOR DENTAL SPLINTING**

Only for professional use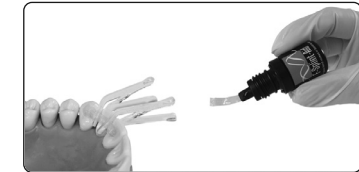

Indicated for splinting of traumatically or periodontally loose anterior teeth, re-implantation of totally luxated anterior teeth (original extracted tooth or built up with composite). F-Splint-Aid & Slim is also suitable to reinforce temporary bridges or for post-orthodontic retainers.

- Non-invasive
- Aesthetic and comfortable for the patient
- Reliable, flexible and time-saving
- Outstanding results
- Unique bottle dispenser

**Order Form:**

Description	Quantity
REF 5972 <b>F-Splint-Aid:</b> Fiber-Splint ribbon (4 mm wide, 120 mm long) impregnated with Fiber-Bond + 5 Clip&Splint application clips	
REF 5977 <b>F-Splint-Aid Slim:</b> Fiber-Splint ribbon (2 mm wide, 120 mm long) impregnated with Fiber-Bond + 5 Clip&Splint application clips	
REF 5980 <b>F-Splint-Aid &amp; Slim:</b> REF 5972 + REF 5977	
REF 5975CS <b>Clip&amp;Splint, autoclavable application clips, 30 pcs</b>	
REF 5991 <b>Fiber-Etch, Bond &amp; Flow:</b> 1 Fiber-Etch (syringe 3g + 10 tips) + 1 Fiber-Flow (syringe 2g + 10 tips) + 1 Fiber-Bond (bottle 3ml)	

Address Dental Dealer: \_\_\_\_\_

Signature and Date: \_\_\_\_\_

Stamp: \_\_\_\_\_